




ALARM PILL BOX WITH VIBRATION

Model: 202V

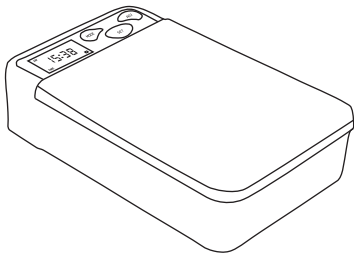


INSTRUCTION MANUAL

Congratulations!

You're the owner of one of the most effective Alarm Pill Box available today. The Pill Box has been designed to Wake-up for the first time of the day when you take your pills. Remind you to take your pills at hourly intervals, and disable the alarm after go to sleep. The alarm gives beep alarm and/or vibration option.

Features

- Multiple pill compartments for each day or different pills
- LCD display for time and alarm indicator
- Alarm Beep sound, visual reminder and vibration alarm option
- Wake-up and sleep function to work with your daily activities
- T1 and T2 two time intervals option
- Anti-humidity pill box
- Powered by one "AAA" battery 1.5V DC.

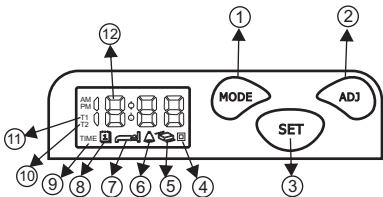

1. Mode Button
2. Adjust Button
3. SET Button
4. Alarm Disable Icon
5. Vibration Icon
6. Alarm Icon
7. Sleep Time Icon
8. First Alarm Time Icon
9. " Time " indicate for real time mode
10. T2 Interval Alarm
11. T1 Interval Alarm
12. Time Display

Installing and Replacement Battery

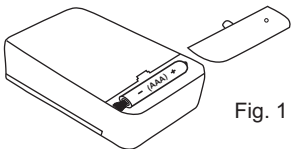


Fig. 1


1. Open battery cover.
2. Insert one "AAA" battery as shown in Fig. 1. Be sure to install with correct polarity as indicate inside the battery compartment.
3. Replace the battery cover.
4. When the alarm sound getting weak or LCD dim to replace a new "AAA" battery.

Timer & Alarm Setting




When first time install or replace a new battery, it will enter the setting mode automatically. Below is a quick setting road map and procedure.

Quick Setting Road Map

Timer Setting

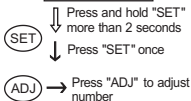


Alarm Setting

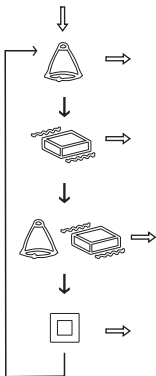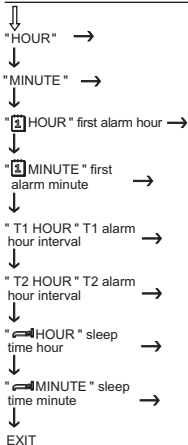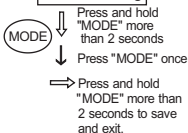


Timer Setting Procedure

1. Press and hold "SET" button until hour digit is flashing, now it is under setting mode. (If there is no any entry more than 30 seconds, it will exit to time mode)
2. Press "ADJ" button to adjust the current "Hour" digit with correct Am or Pm.
3. Press "SET" button to shift to "Minute" digit flash.
4. Press "ADJ" button to adjust the current "Minute" digit.
5. Press "SET" button the icons "1" and "Hour" digit flashing for setting the wake up first time take pills time in each day.
6. Press "ADJ" button to adjust the desired "Hour" digit with correct Am or Pm.
7. Press "SET" button the icons "1" and "Minute" digit flashing.
8. Press "ADJ" button to adjust the desired "Minute" digit. (It is increment







with 10 minutes each step of key entry)

9. Press "SET" button the icons "T1" and "Hour" digit is flashing for setting the first interval timer for alarm.
10. Press "ADJ" button to adjust the desired T1 interval from 1 up to 12 hours.
11. Press "SET" button the icons "T2" and "Hour" digit is flashing for setting the second interval timer for alarm. (If you don't need the second interval timer alarm simply skip step (12) to press "SET" again to keep hour at "00" for not activated T2 alarm)
12. Press "ADJ" button to adjust the desired T2 interval from 1 up to 12 hours.
13. Press "SET" button the icons " and "Hour" digit is flashing for setting the time of sleep. (i.e. after the sleep hour, the alarm

is disabled until next day first time alarm)




14. Press "ADJ" button to adjust the desired sleep "Hour" digit with correct Am or PM.
15. Press "SET" button the icons " and "Minute" digit is flashing.
16. Press "ADJ" button to adjust the desired sleep "Minute" digit. (It is increment with 10 minutes each step of key entry)
17. Press "SET" button complete the setting and exit to time mode.

Alarm Setting Procedure

1. Press and hold "MODE" button until the " " icon flashing to enter the alarm setting mode. (If there is no any entry more than 30 seconds, it will exit to time mode)
2. Press "MODE" button to select one of the alarm mode below:
" " (alarm) , " " (vibration),
" " (alarm & vibration), or
" " (disable).

3. Press and hold "MODE" button until icon stopped flashing to exit alarm setting mode.

NOTE:

- ① While Alarming you may press "MODE" to stop the alarm, but the T1 or T2 icon will continue to flashing up to 30 minutes to remind you which interval timer is set off. During the visual reminder flashing period all setting change function is inhibited.
- ② When "   " is selected, the alarm beep sound and vibration is alternated during the alarm cycle.
- ③ "  " icon is set the alarm totally inactive, must set to one of the 3 alarm modes for active the pill alarm function.

Enter Sleep Mode

The sleeping mode is to set the pill box off for power saving, while you don't use it .

- * Press and hold "ADJ" button until the LCD is off to enter the sleeping mode.

Wake up from Sleep Mode

- * Press and hold "SET" button until the LCD is turn on and "Hour" digit is flashing to wake up device. Please follow the "Timer Setting" and "Alarm Setting" to reset the time and alarm.

Specifications

- * Operation Voltage:
"AAA" battery X 1, 1.5V DC
- * Alarm time:
60 seconds beep, and visual indicator 30 minutes
- * Type of material: PP

611-202V00-000
VER 1.0,04/2006