

Troubleshooting

If you have trouble while using the device, please check the following points


Error	Possible cause	How to correct
Can't normally message	No battery	Install battery
	Battery worn out	Replace new battery
	Incorrect Battery polarity installation	Install battery in the correct polarity

Caution

- * Do not use device for infants
- * Do not disassemble, repair, or remodel the main unit
- * Do not use the device for purpose other than massager
- * Do not drop device and avoid suddenly shock

SLIMSTAR ELECTRONIC MASSAGER

Model: 604

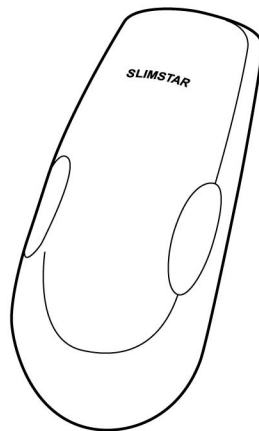

INSTRUCTION MANUAL

Thank you for purchasing SLIMSTAR MASSAGER. This device can relax pain of the muscle and slimming aid, to activate easier for fat burning. This device designed with 2 "AA" batteries operation and compact size easy to use anywhere at anytime


Features of the Product

- * Portable and Handheld stylish easy to use
- * Two different massager pin arrays
- * Relaxes after exercise
- * Special slimming aid fit for arm, leg, hips, belly, waist, wrist, neck and shoulder
- * Power Source: Battery "AA" x 2

Battery Installation and Replacement

1. Press and Pull upward on the opener clip (as Fig.1) to open the battery cover
2. Install 2 "AA" size batteries into battery compartment make sure in correct polarity direction
3. Replace the battery cover
4. While massager vibration weak, Please replace new batteries

Note: Remove the batteries if the device will not be used for an extended period of time

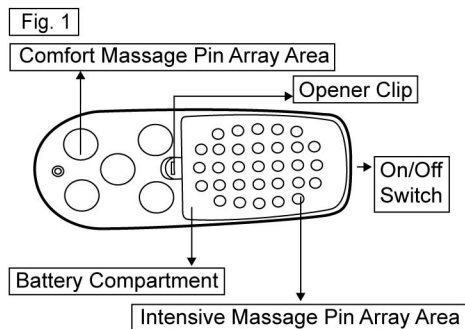

1. Turn switch "ON" to start vibration massage
2. Firmly hold device and apply with an adequate pressure to the portion of body for comfort massage
3. Suggest slowly moving device back and forth across the massage area for best comfort result

Specification

1. Item: Slimstar Electronic Massager
2. Model: 604
3. Power Source: 1.5V "AA" x 2 alkaline batteries
4. Weight Approximately 140g
5. Size: 145 x 62 x 49mm
6. Shake Frequency: 6000Hz