

EYE CARE MASSAGER

Model: 605E



INSTRUCTION MANUAL

TABLE OF CONTENTS

Introduction.....	2
Notes on Safety.....	2
Names of Parts.....	3
Features.....	4
Method of Usage.....	4
Troubleshooting.....	6
Product Care and Storage.....	6
Specifications.....	7

INTRODUCTION

Thank you for purchasing the Eye Care Massager. Before you begin The device may improve the reduction in the appearance of fine lines, dark rings around the eyes, eye bags, and relief from eye tiredness. Before you begin using your new massager, please read this instruction manual thoroughly for the correct methods of usage, noting especially the section on SAFETY.

This manual should be kept on hand for future reference.


NOTES ON SAFETY

- * Do not use this product if you have an eye operation recently.
- * Stop using the product and consult your doctor if you are unwell, or if use irritates the skin.
- * Stop using the product if you feel any physical discomfort or if you detect anything unusual in the performance of the massager.
- * Do not wear spectacles or contact lenses while using the product.
- * Children under 8 years old must be guided by an adult while using the product.
- * It is advisable not to use the product for more than 15 minutes continuously.
- * Use alkaline batteries for optimum performance.
- * Do not use in a room with high humidity or while

showering or bathing.

- * Avoid using in strong sunlight and near corrosive gas.
- * Do not use while walking outdoors or driving a car.
- * Do not sit or step on the device.
- * Do not remove the Magnetic Stone when the massager is being used.
- * Do not pull the Eye Massager Rubber cushion.
- * Do not handle the Eye Massager with wet hands.
- * Do not drop the product on the ground or dip it into water.
- * Use "AAA" size batteries. remove batteries from the compartment when not using long time so as to prevent any battery leakage.

NAMES OF PARTS

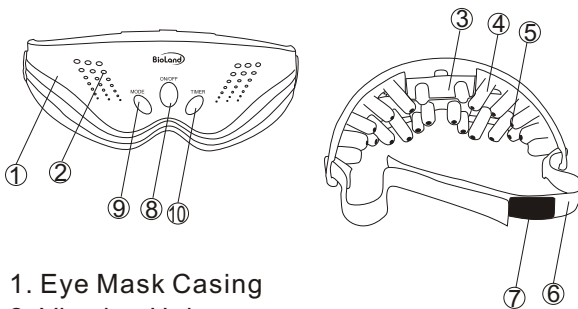

1. Eye Mask Casing
2. Viewing Hole
3. Battery compartment ("AAA" X 2 batteries)
4. Rubber Cushion
5. Magnetic Nodes

6. Elastic Strap
7. Velcro
8. Power On/Off button
9. Mode button
(3 Modes - Continuous Vibration, Wave Pulse and Medium Pulse)
10. Timer button
(5 minutes/one beep, 10 minutes/2 beeps, 15 minutes/3 beeps)

FEATURES

- * Eye Care Massager Relieve Eye Fatigue from Work.
- * 3 Modes Vibration, Adjust to your Liking.
- * Silicon - Rubber Massage Fingers with Magnetic.
- * Magnetic Nodes to improve circulation around the eye area.
- * 5, 10 and 15 minutes Auto Shut - off Timer.
- * Power - "AAA" X 2 batteries.


METHOD OF USAGE

Make sure to install "AAA" X 2 batteries into the battery compartment.

1. Press "Power" button to turn the device On.


2. The device will start vibration at the preset state (Mode: Continuous Vibration, Timer: 5 Minutes), or your last operation Mode and timer state.

3. Press "Mode" button to change the mode in following sequence -
Continuous Vibration → Medium Pulse → Wave Pulse



4. Press "Timer" button to change the timer in following sequence-

5 Minutes(one Beep) → 10 Minutes(2 Beeps) → 15 Minutes(3 Beeps)



5. Press "Power" button again to turn device Off, or when timer is up with one long beep and turn off automatically

!! Caution

- * The Rubber Cushion should be cleaned before use. The Magnetic Nodes can be removed for cleaning by pressing the tip of the Rubber Cushion.
- * It is important to ensure the Eye Care Massager is worn properly and that the Elastic Strap is of the correct tension.
- * Remove the Eye Care Massager once the massage is completed, close your eyes and rest for 1-3 minutes. This is to allow your eyes to recover their normal vision.

TROUBLESHOOTING

Do not disassemble or attempt to repair the device yourself.

Conditions	Possible Causes	Remedy
No Power	* Are the batteries weak?	* If yes, replace them with new batteries.
	* Are the batteries installed correctly?	* Ensure the polarities of batteries are correct * Ensure the batteries are firmly affixed in the battery compartment.
The Mode cannot be selected	* Are the batteries weak?	* If yes, replace them with new batteries. * Ensure the batteries are firmly affixed in the battery compartment.
The Timer cannot be programmed	* Are the batteries weak?	* If yes, replace them with new batteries. * Ensure the batteries are firmly affixed in the battery compartment.

PRODUCT CARE AND STORAGE

- * Turn off the power before cleaning.
- * Clean the unit with a damp cloth or a neutral cleanser and then wipe with a dry cloth. Do not use volatile

liquids such as Benzene, thinner or gasoline for cleaning.

- * Remove and clean the Magnetic Nodes by press the tip of the Rubber Cushion.
- * Do not place the unit in areas of high temperature, high humidity or in direct sunlight. Store the unit where there is no moisture.
- * Do not stack heavy items on the device.
- * Remove the batteries when the unit is not being used for extended period of time.

SPECIFICATIONS

Model name: Eye Care Massager

Model number: 605E

Frequency of Vibration: 40Hz - 100Hz

Power Consumption: Less than 2 W

Power Supply: 3V DC ("AAA" X 2 batteries)

611-605E00-000
VER 1.0, 10/2006